

Instructions For Use



Helping you keep track of your insulin doses

TABLE OF CONTENTS

	P.
1. SYMBOLS IN THIS INSTRUCTIONS FOR USE.....	4
2. WARNING & CAUTION	5
3. STORAGE	5
4. DIRECTIONS FOR USE.....	6
4.1 FEATURES.....	6
4.2 INTENDED USE	6
5. MALLYA COMPATIBILITIES WITH PENS.....	7
6. GETTING TO KNOW YOUR MALLYA DEVICE	8
6.1 GETTING STARTED	8
6.2 FEATURES OF YOUR MALLYA.....	9
6.3 WEIGHT AND SIZE OF THE MAIN ELEMENTS	9
6.4 STORAGE INFORMATION	9
6.5 TRANSPORT AND PACKAGING	10
7. INSTALLING THE MALLYA APPLICATION.....	10
7.1 TECHNICAL COMPATIBILITIES	10
7.2 DOWNLOAD THE MALLYA APPLICATION AND GET STARTED.....	11
7.2.1 Select "Get started":	11
7.2.2 A video will guide you through the unboxing steps. Select "Continue" to watch the Quick Start video:.....	11
8. CARING FOR YOUR MALLYA.....	12
8.1 CHARGING YOUR MALLYA	12
8.1.1 Plug the powered USB cable into the Mallya micro-USB port.	12
8.1.2 Check that the Mallya LED is slowly flashing green.	12
8.1.3 Let the battery charge for at least 10 minutes.	12
8.2 CALIBRATING YOUR MALLYA	13
8.2.1 Unplug the Mallya smart base from the USB cable.....	13
8.2.2 Check that the Mallya LED is flashing orange.	13
8.2.3 Slowly tilt the Mallya smart base side to side to CALIBRATE.....	13
8.2.4 Continue tilting until:	13
8.3 ATTACHING YOUR MALLYA TO PEN.....	14
8.3.1 ATTACHING SMART BASE TO PEN	14
8.3.1.1 Hold the Mallya smart base so that the clear square plastic window is facing you	14
8.3.1.2 Align the Mallya smart base clear square plastic window with the pen dose window, then snap the smart base onto the pen.....	14
8.3.1.3 Make sure the Mallya smart base window is aligned with the pen dose window.	14
A pen symbol is represented on the side of the Mallya smart base to help you position the pen in the correct orientation.	14
8.3.2 ATTACHING BUTTON TO PEN.....	14
8.3.2.1 Place and insert the Mallya button over your pen injection button. Make sure the button is properly attached.	14
8.3.2.2 The Mallya smart base LED will flash orange and green when the button and smart base are properly mounted onto the pen.....	14
8.4 PAIRING YOUR MALLYA	16
8.4.1 Make sure the Mallya smart base is within 12 inches of your smartphone and select "Start pairing":	16
8.4.2 Select "Pair"	16
8.4.3 Choose the insulin pen model onto which the Mallya device is assembled.....	16
8.4.4 Confirm the pen model currently assembled with the Mallya device.	16
8.4.5 Well done! Your Mallya device is now paired with the Mallya application on your smartphone.	17
8.4.6 The injection screen is displayed.	17
9. USING YOUR INSULIN PEN WITH MALLYA	18
9.1 SECURITY TEST - PRIMING	18
9.1.1 Follow the injectable insulin pen Instructions for Use to carry out the priming step.	18
9.1.2 At the end of the priming step, keep the injection button pressed down:.....	18
- While Mallya's green LED is flashing,	18
- Until Mallya emits an audible signal (BEEP) signaling the end of the priming step.	18
9.1.3 You can release the pressure on the button when the green LED stops flashing and the "BEEP" signaling the end of priming is emitted.....	18
9.2 INJECTION	19
Check the insulin dose to inject as requested by the drug manufacturer's requirements.	19

	Turn the dose knob to select the number of units to inject. The dose indicator should line up with the dose to inject.	19
	Press on the injection button with your thumb until you return to ZERO (complete dose injection)	19
	Keep the injection button pressed in and the needle in the skin :	19
	- While the Mallya's green LED is flashing,	19
	- Until Mallya emits an audible signal (BEEP) signaling the end of the injection.	19
	You can release the pressure on the button and then remove the needle from the skin when the green LED stops flashing and the "BEEP" signaling the end of injection is emitted.	19
10.	MANAGING RECORDED DATA	20
10.1	DISPLAY OF THE RECORDED DATA	20
10.2	MANAGE YOUR RECORDED DATA	21
10.2.1	You can change a «priming» or a «dose» quantity already recorded.....	21
10.2.2	You can change an injection recorded as a «dose» to a «priming», or a «priming» to a «dose»	21
10.2.3	You can delete a «dose» or a «priming»	22
10.2.4	You can add an injection that was not recorded to the history.....	22
10.2.5	You can fill in a dose that was not captured by the device	23
11.	OTHER FEATURES OF THE MALLYA APPLICATION	23
11.1	GENERATE A REPORT	23
11.2	PAIR TWO MALLYA TO YOUR MOBILE APP	24
11.3	SETTINGS	25
11.4	"HELP" SCREEN	26
12.	TRANSFERRING MALLYA TO A NEW PEN	27
12.1	REMOVE MALLYA FROM THE EMPTY PEN	27
12.1.1	Remove the Mallya smart base from the insulin pen. Use your thumb to press on Mallya and to remove it from the pen.....	27
12.1.2	Then, remove the Mallya knob button from the pen.....	27
12.2	ATTACH YOUR MALLYA TO A NEW PEN	27
12.2.1	Injection screen before transferring Mallya onto a new pen:	28
12.2.2	Remove Mallya from the empty pen / attach Mallya onto a new pen (smart base and / or button).....	28
12.2.3	On the next PRIMING or an INJECTION:	28
12.3	COLLECT YOUR INJECTION DATA FROM MALLYA AFTER SMARTPHONE DISCONNECTION	30
13.	TROUBLESHOOTING	31
13.1	TROUBLESHOOTING MALLYA WITH SMART BASE SIGNALS	31
13.2	TROUBLESHOOTING MALLYA WITH THE APPLICATION	33
13.3	RESET INSTRUCTIONS	35
	Disassemble Mallya as indicated <i>step 8.1</i>	35
	Flip Mallya over so that you can see the following indications:.....	35
	Take the Reset Key:	35
	Insert the Reset Key into the hole shown below and WAIT until 3 beeps are emitted before removing the key.	35
	Then follow <i>step 4</i> . to prepare your device for a new operation.	35
14.	MAINTENANCE AND INSPECTION	36
14.1	MALLYA DEVICE LIFETIME	36
14.2	MALLYA APPLICATION UPDATES	36
14.3	CLEANING YOUR MALLYA	36
14.4	MAINTAIN WATER, DUST AND LIGHT RESISTANCE	36
15.	TECHNICAL SPECIFICATIONS	37
15.1	COMPLIANCE WITH STANDARDS / REGULATION AND CLASSIFICATIONS	37
15.2	SMART BASE LABELLING	37
15.3	SYMBOLS ON THE DEVICE AND ITS PACKAGING	38
15.4	ELECTRICAL RATINGS	39
15.5	MATERIALS	39
15.6	ACCURACY	39
15.7	DISPOSAL	40
16.	GENERAL INFORMATION	41
17.	DISCLAIMER	41

1. SYMBOLS IN THIS INSTRUCTIONS FOR USE

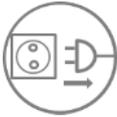
Symbol	Meaning	Signification
	WARNING	Improper operation may result in serious injury* ¹ or death to the user, patient
	CAUTION	Improper operation may result in bodily injury* ² or property damage* ³
	CAUTION	Disconnect the device from power supply before servicing/cleaning
	NOTE	Important information for operation
	CAUTION	Consult the Instructions for Use documents
	CAUTION	Do not use if the packaging is damaged

Table 1 - Symbols used in this IFU (Instructions for Use)

*1 Serious injury means electrical shock or poisoning that causes a subsequent complication or requires hospitalization or long-term outpatient treatment.

*2 Bodily injury means an injury, burn, electrical shock and so on that will not necessitate hospitalization or long-term outpatient treatment.

*3 Damage to property means extensive damage to a house and/or household goods as well as a domestic animal and pet.

2. WARNING & CAUTION

NOTES:

- Mallya must only be used for the intended use described in this Instructions for Use document.
- Read the Quick Start Guide and these Instructions for Use before you use Mallya for the first time. Keep these instructions in a safe place.
- Warning and Caution statements are included throughout this document.



WARNING: improper operation may result in serious injury or death

- If you are not certain that you injected your insulin, do not start or repeat your injection. Monitor your blood glucose as instructed by your healthcare provider.
- Disconnect the USB cable from the USB charging port and disconnect Mallya from the USB cable before using it.
- Mallya must only be paired with a compatible mobile application.
- Mallya has been designed to work with ONE insulin pen. If you unpair your Mallya smart base from the insulin pen, you MUST reselect your insulin pen.
- Do not disassemble, modify or repair the Mallya by yourself. All servicing should be done by a BIOCORP PRODUCTION authorized dealer.
- Do not try to access the internal hardware of your device.
- Never install a mobile application from unofficial stores on your phone.
- Jailbreak or rooted phone exposes it to dangers and will invalidate the Mallya warranty.



CAUTION: improper operation may result in bodily injury or property damage

- The Instructions for Use should be accessible to you at all times and you should read all instructions carefully before using the device.
- Mallya is intended for Over the Counter use and is not a prescription device.
- Mallya is supplied with a USB cable. Use the cable supplied with the device to connect it to a USB power port. Do not use any other cable. Other cables may create problems or interfere with other electronic devices.
- Protect your personal health data by using a password on your smartphone.
- If the device is bumped or dropped, it may malfunction. If it is malfunctioning, please contact a BIOCORP PRODUCTION authorized dealer for technical support.
- Use only neutral cleaning agents. Do not wipe the surface of Mallya with other chemical products or solvents (e.g. acetone) as they can lead to discoloration or deterioration.
- Before disposing of Mallya: reset the device to delete all your personal health data (see **step 9.3.**)

3. Storage

Store the Mallya at room temperature between 59°F to 95°F (15°C to 35°C).

Keep Mallya away from:

- Water
- Pacemakers and defibrillators
- Metallic parts
- Small children. Your Mallya contains small parts that may cause injury or become a choking hazard.

4. DIRECTIONS FOR USE

4.1 FEATURES

Mallya is composed of a smart base and a button that can be mounted on a disposable insulin pen.



Mallya allows you to:

- record the injected selected increment (insulin dose),
- record the day and time when each insulin dose was injected,
- transmit the dose, day/time when each insulin dose was injected to a compatible mobile app. on a bluetooth paired mobile device (smartphone).

The compatible mobile application is then able to display the injected Insulin Selected Dose (ISD) by day/time.

4.2 INTENDED USE

Indications for use:

Mallya can be used either for **type I or type II diabetes**.

The device is intended **to assist patients maintain accurate records of their insulin usage**.



WARNING

The device should not be used by children and women with gestational diabetes.

Contraindications:

There are no known contraindications to the use of Mallya.

Intended Patient population:

Intended users are **patients and medical professionals**.

Mallya can only be used by patients already familiar with the pen injectors: **Kwikpen®** (Eli Lilly), **Solostar®** (Sanofi) or **Flexpen®** (Novo Nordisk).

Over the Counter (OTC) Medical Device:

Mallya is intended for **Over the Counter use** and is not a prescription device.

Sterility information:

Mallya is intended to be non-sterile.

Adverse reaction information:

There are no known adverse reactions to the use of Mallya.

Intended use location:

The device is intended for use in a **medical office or at home**.

5. MALLYA COMPATIBILITIES WITH PENS



WARNING


Make sure your Mallya is compatible with your pen.

MALLYA IS COMPATIBLE WITH THE PENS WITH THE FOLLOWING MOLECULES AND CONCENTRATIONS:					
Mallya for FLEXPEN®		Mallya for KWIKPEN®		Mallya for SOLOSTAR®	
International non-proprietary name	Concentration	International non-proprietary names	Concentration	International non-proprietary names	Concentration
INSULIN DETEMIR	100 IU/mL	INSULIN GLARGINE	100 IU/mL	INSULIN GLARGINE	100 IU/mL
INSULIN ASPART	100 IU/mL	INSULIN LISPRO	100 IU/mL	INSULIN GLARGINE	300 IU/mL
INSULIN ISOPHANE HUMAN BIOGENETIC	100 IU/mL	INSULIN LISPRO	200 IU/mL	INSULIN LISPRO	100 IU/mL
		INSULIN ISOPHANE HUMAN BIOGENETIC	100 IU/mL	INSULIN GLULISINE	100 IU/mL
				INSULIN GLARGINE AND LIXISENATIDE	100 IU/mL + 33 mcg/mL
WARNING					
EXCLUDED VERSIONS: (MALLYA IS NOT COMPATIBLE WITH THE FOLLOWING INSULIN PENS)					
		Mallya for KWIKPEN®		Mallya for SOLOSTAR®	
		HUMALOG® JUNIOR		TOUJEO® MAX	

Table 2 - Mallya compatibilities with pens

6. GETTING TO KNOW YOUR MALLYA DEVICE

6.1 GETTING STARTED

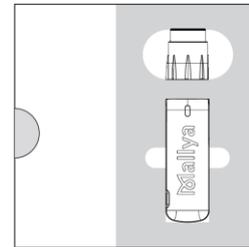

The shipment box contains the following items. The Part Numbers (PN) vary depending on the Mallya model (Mallya designed for FLEXPEN or KWIKPEN or SOLOSTAR):




Mallya DEVICE:		
1 X Mallya SMART BASE		
 Applied part	For the insulin pen FLEXPEN® NOVO NORDISK : PN: 0273778 For the insulin pen KWIKPEN® ELI LILLY : PN: 0273694 For the insulin pen SOLOSTAR® SANOFI : PN: 0273777	
2 X Mallya BUTTONS		
 Applied part	For the insulin pen FLEXPEN® NOVO NORDISK : PN : 0273727	
 Applied part	For the insulin pen KWIKPEN® ELI LILLY : PN : 0273698	
 Applied part	For the insulin pen SOLOSTAR® SANOFI : PN: 0273710	
Mallya ACCESSORIES:		
1 x USB CABLE	1 x RESET KEY	1 x QUICK START GUIDE
		 CAUTION
Part Number: B014JOOZFS	Model: B004OQOQQ0 Manufacturer: Bright Sea Industrial. Co	Instructions For Use (reference: IFU_MAL_US) Quick Start Guide (reference: QSG_MAL_US)

Table 3 - Mallya box content

NOTE :

Before you use your Mallya for the first time, you have to charge it. Use the provided USB cable to connect the Mallya smart base to a USB port or outlet. Let it charge for at least 10 minutes.

6.2 FEATURES OF YOUR MALLYA

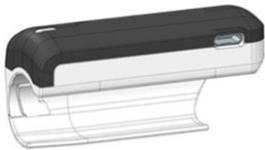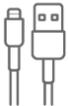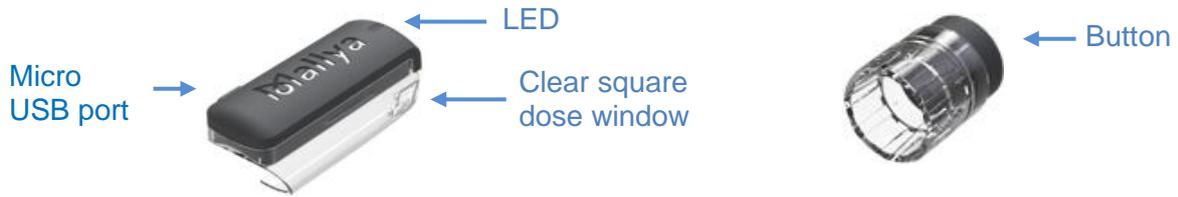

6.3 WEIGHT AND SIZE OF THE MAIN ELEMENTS

Element	Weight	Size
Mallya smart base	0.016 kg	KWIKPEN® smart base: (60.8x28.9x21.3) mm FLEXPEN® and SOLOSTAR® smart bases: (60.8x26.9x21.3) mm
Mallya button	0.006 kg	KWIKPEN® button: (11.6 x Ø22) mm SOLOSTAR® button: (24.4 x Ø20) mm FLEXPEN® button: (26.3 x Ø19.3) mm
Mallya total weight	0.022 kg	

Table 4 - Weight and size of the main elements

6.4 STORAGE INFORMATION

	Temperature	Atmospheric pressure	Relative humidity
Transport	-10°C to +40°C (14°F < T° < 104°F)	500 hPa to 1060 hPa	10% to 90%
Storage	-10°C to +40°C (14°F < T° < 104°F)	800 hPa to 1060 hPa	10% to 90%
Operating	+15°C to +35°C (59°F < T° < 95°F)	800 hPa to 1060 hPa	30% to 70%

Table 5 - Storage information

6.5 TRANSPORT AND PACKAGING

- The device must be transported and stored in its original packaging. Storage and transport conditions need to meet conditions described in the previous **step 2.4**.
- Keep the original packaging in case of return or transport of device.
- Check if there is any damage to the packaging. If the packaging is damaged, there is a possibility that the device is also damaged. Please notify the carrier if any damage is found and report it to the manufacturer.



CAUTION:

Do not use Mallya if the packaging is damaged.

- Leave the device in a room for 30 minutes before unpacking to ensure there is no condensation.

NOTE :

Keep all original packaging for future use. The system must always be transported in its original packaging specially designed to protect it against damage.

7. INSTALLING THE MALLYA APPLICATION

Mallya connects your insulin pen to your smartphone!

Use the Mallya smartphone application to keep track of your insulin doses.

7.1 TECHNICAL COMPATIBILITIES

The list of smartphones validated by BIOCORP and compatible with the Mallya application Version 1.7.0 is shown below:

SMARTPHONE MODEL	COMPATIBLE OPERATING SYSTEM
Samsung Galaxy S9	Android 10.0
Samsung Galaxy S8 Plus	Android 9.0
Samsung Galaxy S6	Android 7.0
Honor 10	Android 8.1
Huawei P20 Lite	Android 8.0
Huawei Mate 10 Lite	Android 8.0
LG V30	Android 8.0
iPhone SE	iOS13
iPhone 6s	
iPhone 6s Plus	
iPhone 7	
iPhone 7 Plus	
iPhone 8	
iPhone 8 Plus	
iPhone X	
iPhone XR	
iPhone 11	

Table 6 - Mallya application technical compatibilities



CAUTION

This list represents the list of smartphones and associated operating systems that have been tested and validated by BIOCORP. This list is not exhaustive.

7.2 DOWNLOAD THE MALLYA APPLICATION AND GET STARTED

Download the Mallya application from:




Once installed, select the following icon to open up the application:



The first time you open the application, the Mallya application terms of service screen is displayed. Please read the Mallya application terms of use before accepting them and confirming your choice.
before accepting them and confirming your choice.

The privacy policy screen is then displayed: make sure you understand it before accessing the **“Get Started”** screen.
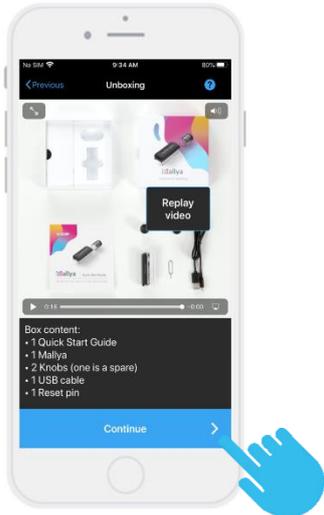
<p>7.2.1 Select "Get started":</p> 	<p>7.2.2 A video will guide you through the unboxing steps. Select "Continue" to watch the Quick Start video:</p> 
---	--

The next screens will guide you through the 4 operations required to start up your Mallya:

- Switch on (as explained in step **4.1. Charging your Mallya**)
- Calibrate (as explained in step **4.2. Calibrating your Mallya**)
- Assemble (as explained in step **4.3. Assembling your Mallya to the insulin pen**)
- Pair (as explained in the following step **4.4. Pairing your Mallya**)

8. CARING FOR YOUR MALLYA

8.1 CHARGING YOUR MALLYA

NOTE:

Mallya must be charged before first-time use.

**CAUTION**

- Mallya is supplied with a USB cable. Do not use any other cable. Use the cable supplied with the device to connect it to a USB power port.
- You cannot use your Mallya for an injection while it is connected to a USB power port and charging.

8.1.1 Plug the powered USB cable into the Mallya micro-USB port.



8.1.2 Check that the Mallya LED is slowly flashing green.

8.1.3 Let the battery charge for at least 10 minutes.

**NOTES:**

- Fully recharging the battery takes approximately 2 hours. When the battery is fully charged, the **green** LED will turn off.
- Optimal autonomy is reached when Mallya's battery is fully charged. You can then use Mallya for at least 2 weeks.

8.2 CALIBRATING YOUR MALLYA
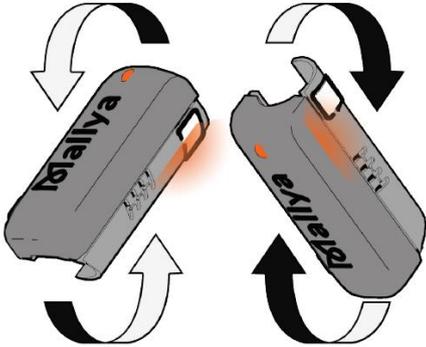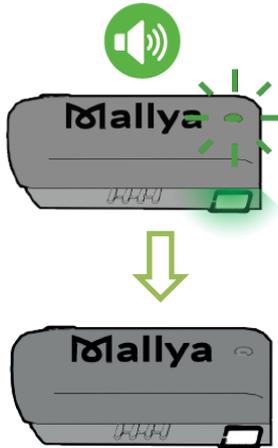
<p>8.2.1 Unplug the Mallya smart base from the USB cable.</p>	
<p>8.2.2 Check that the Mallya LED is flashing orange.</p>	
<p>8.2.3 Slowly tilt the Mallya smart base side to side to CALIBRATE.</p>	
<p>8.2.4 Continue tilting until:</p> <ul style="list-style-type: none"> ✓ The smart base beeps and the LED light turns green. ✓ Then the light will turn off. 	

8.3 ATTACHING YOUR MALLYA TO PEN



CAUTION

The Mallya smart base and button must be correctly attached onto the pen, otherwise, data will not be transferred.

8.3.1 ATTACHING SMART BASE TO PEN

<p>8.3.1.1 Hold the Mallya smart base so that the clear square plastic window is facing you</p>	
<p>8.3.1.2 Align the Mallya smart base clear square plastic window with the pen dose window, then snap the smart base onto the pen.</p>	
<p>8.3.1.3 Make sure the Mallya smart base window is aligned with the pen dose window.</p>	

NOTE:

A pen symbol is represented on the side of the Mallya smart base to help you position the pen in the correct orientation.



8.3.2 ATTACHING BUTTON TO PEN

<p>8.3.2.1 Place and insert the Mallya button over your pen injection button. Make sure the button is properly attached.</p>	
<p>8.3.2.2 The Mallya smart base LED will flash orange and green when the button and smart base are properly mounted onto the pen.</p>	

**CAUTION**

The Mallya smart base LED remains **orange** if:

- the clear square plastic window of the Mallya smart base and the dose window of the pen are not correctly aligned,
- or, the Mallya knob is not pushed on enough.



In this case, it is necessary to:


- Re-adjust (re-align) the windows,
- or, push on the knob, until the Mallya smart base LED flashes **green** and **orange**.



8.4 PAIRING YOUR MALLYA

To pair the Mallya device to the BIOCORP Mallya application, follow the steps indicated below:


8.4.1 Make sure the Mallya smart base is within **12 inches** of your smartphone and select **“Start pairing”**:

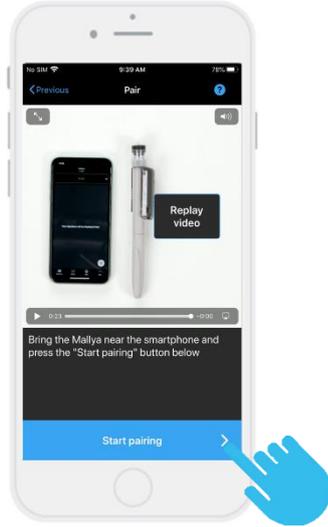


8.4.2 Select **“Pair”**
NOTE: The pairing step may take few seconds.



8.4.3 Choose the insulin pen model onto which the Mallya device is assembled.

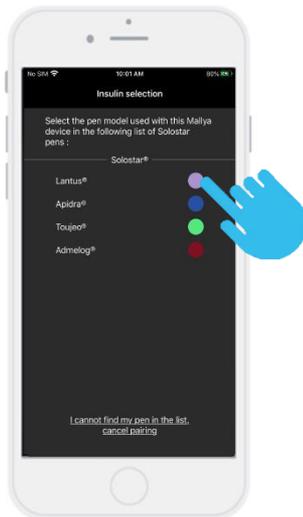


8.4.4 Confirm the pen model currently assembled with the Mallya device.

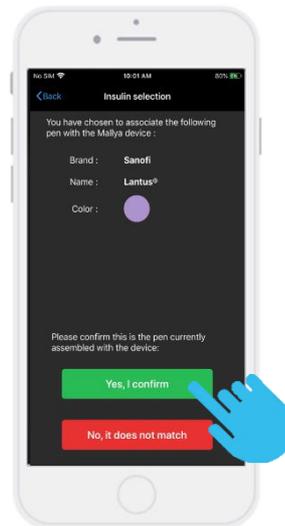

NOTE: In case you cannot find your pen model in the list: select **“cancel pairing”**. The application will guide you to pair a new Mallya.

NOTE: If you select **“No, it does not match”**, the application will return to the previous screen (**step 4.4.3.**) to let you choose another pen model.


8.4.5 Well done! Your Mallya device is now paired with the Mallya application on your smartphone. Select “OK” to move on to the injection screen.

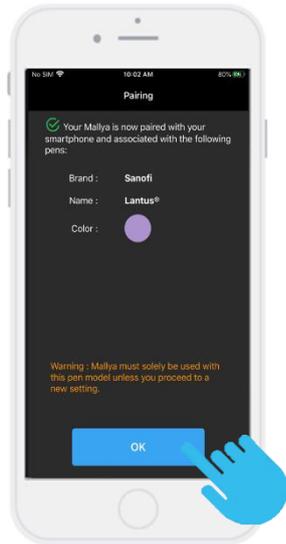

CAUTION

From now on, this Mallya should only be used with this pen model, unless you make a new adjustment.


8.4.6 The injection screen is displayed.

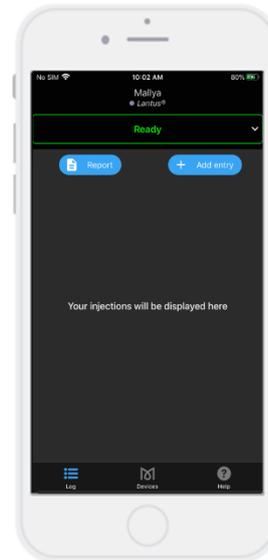

Your insulin pen can now be used as usual and Mallya will track your insulin intake.

NOTE:

When Mallya is paired and correctly attached to the injection pen: the LED is **green**.

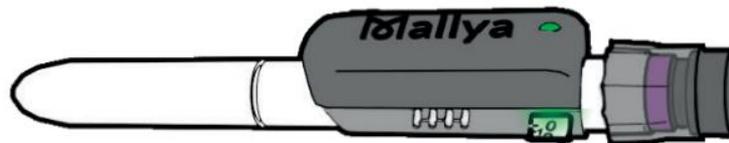

9. USING YOUR INSULIN PEN WITH MALLYA

NOTE :

Once Mallya is attached to the insulin pen, the pen should be used just like a normal insulin pen. Refer to the manufacturer's injector pen Instructions for Use.

9.1 SECURITY TEST - PRIMING



WARNING

Always perform a priming (safety test) before each injection to ensure that:


- ✓ Your pen and the needle are working properly,
- ✓ The right dose of insulin will be injected.

If you do NOT perform the priming, you may receive too much or too little insulin (refer to the manufacturer's injector pen Instructions for Use).

9.1.1 Follow the injectable insulin pen Instructions for Use to carry out the priming step.

9.1.2 At the end of the priming step, keep the injection button pressed down:

- While Mallya's **green** LED is flashing,
- Until Mallya emits an audible signal (BEEP) signaling the end of the priming step.

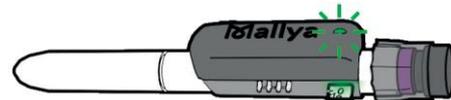

WARNING

If the liquid does not come out of the needle tip during the safety test, do not use the pen and refer to the manufacturer's injector pen Instructions for Use for more information to solve the problem.

9.1.3 You can release the pressure on the button when the green LED stops flashing and the "BEEP" signaling the end of priming is emitted.

The Mallya application will automatically detect a "priming" if:

- You are priming 1 or 2 unit(s), and
- You perform your insulin injection just after the priming step (within 2 minutes)



CAUTION

If the time interval between the two consecutive injections exceeds 2 minutes, the application will automatically detect an injection instead of a priming.

NOTE :

You can always change an injection recorded as a "dose" to a "priming", or a "priming" to a "dose" in case it has not been suitably categorized. Please refer to **step 6.2.2**.

9.2 INJECTION



WARNING

If you are not certain that you injected your insulin, do not start or repeat your injection. Monitor your blood glucose as instructed by your healthcare provider.


Check the insulin dose to inject as requested by the drug manufacturer's requirements.

Turn the dose knob to select the number of units to inject. **The dose indicator should line up with the dose to inject.**

Press on the injection button with your thumb until you return to ZERO (complete dose injection)

Keep the injection button pressed in and the needle in the skin :

- While the Mallya's **green** LED is flashing,
- Until Mallya emits an audible signal (BEEP) signaling the end of the injection.

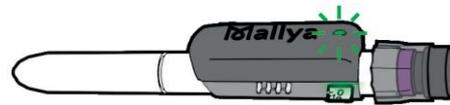

You can release the pressure on the button and then remove the needle from the skin when the **green** LED stops flashing and the **"BEEP" signaling the end of injection is emitted.**



WARNING

If you do not see ZERO ("0") in the pen dose window: it indicates that the full dose has not been injected.

The Mallya application will automatically detect an injection:



The information corresponding to the injected dose is displayed: dose, day and time.

10. MANAGING RECORDED DATA


Once data is transferred from a paired compatible Mallya, you can display and manage any recorded data using the Mallya application as follows:

10.1 DISPLAY OF THE RECORDED DATA

Select the injection screen:



“Doses” or “Priming” injections are then displayed as follows:

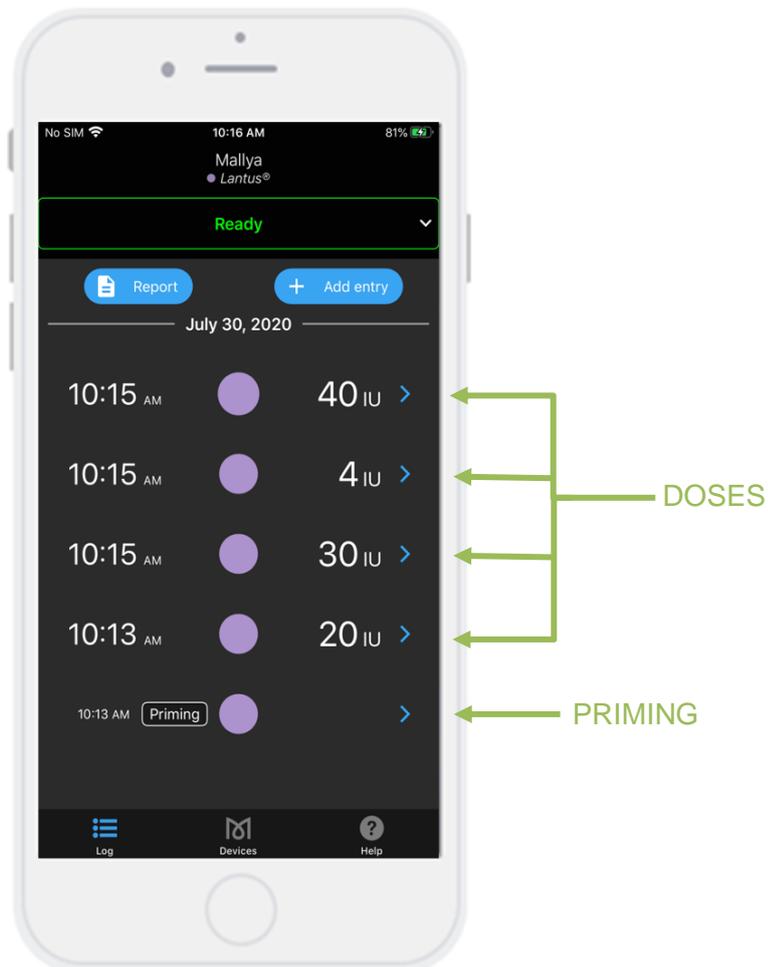

The Insulin Dose information is displayed:


- Type of injection: “**priming**” or “**dose**”
- Injection **day** and **time**
- Insulin dialed **dose** (in International Unit)
- **Color indicator** of the injected insulin type or concentration

10.2 MANAGE YOUR RECORDED DATA


You can manage and edit the following list of items:

- You can change a “priming” or a “dose” quantity (see **step 6.2.1**),
- You can change an injection recorded as a “dose” to a “priming”, or a “priming” to a “dose” (see **step 6.2.2**),
- You can delete a “dose” or a “priming” (see **step 6.2.3**),
- You can add an injection that was not recorded to the history (see **step 6.2.4**),
- You can fill in a dose that was not captured by the device (see **step 6.2.5**).


10.2.1 You can change a «priming» or a «dose» quantity already recorded

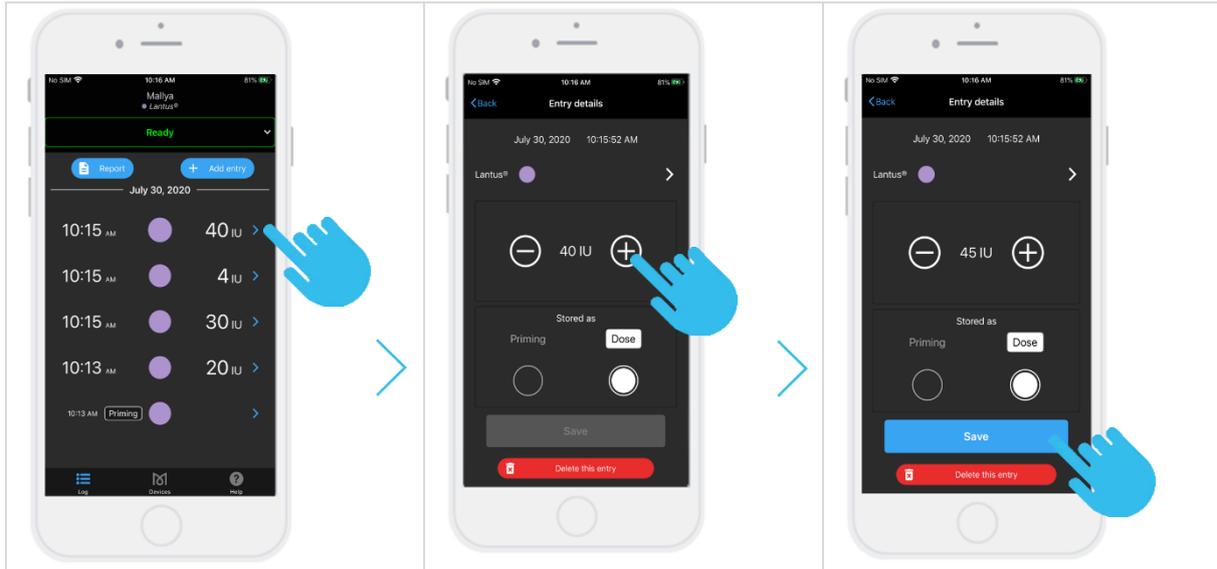

10.2.2 You can change an injection recorded as a «dose» to a «priming», or a «priming» to a «dose»

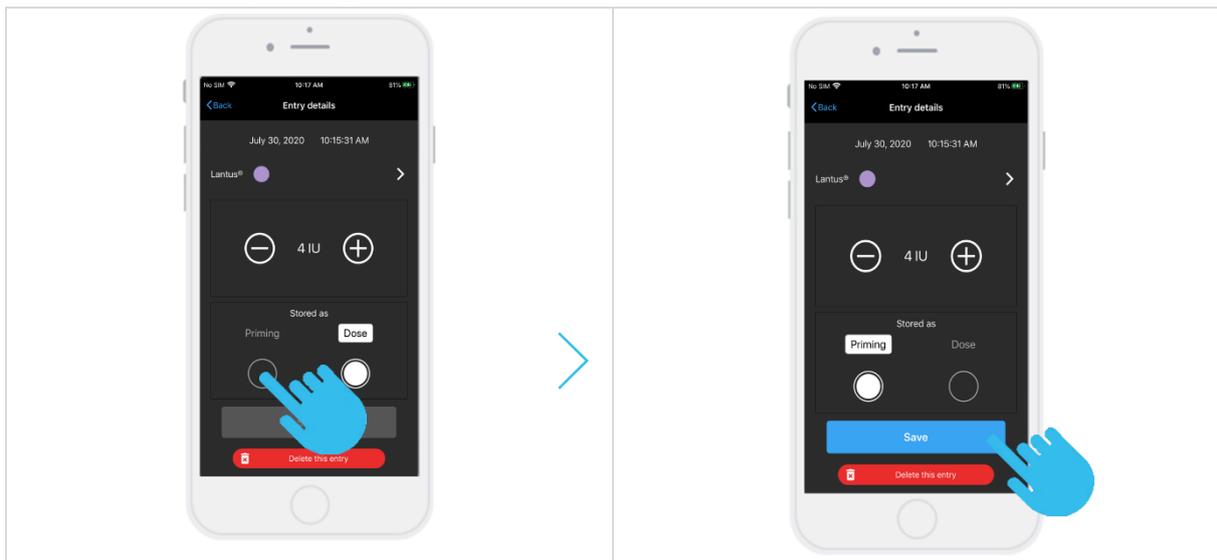


10.2.3 You can delete a «dose» or a «priming»

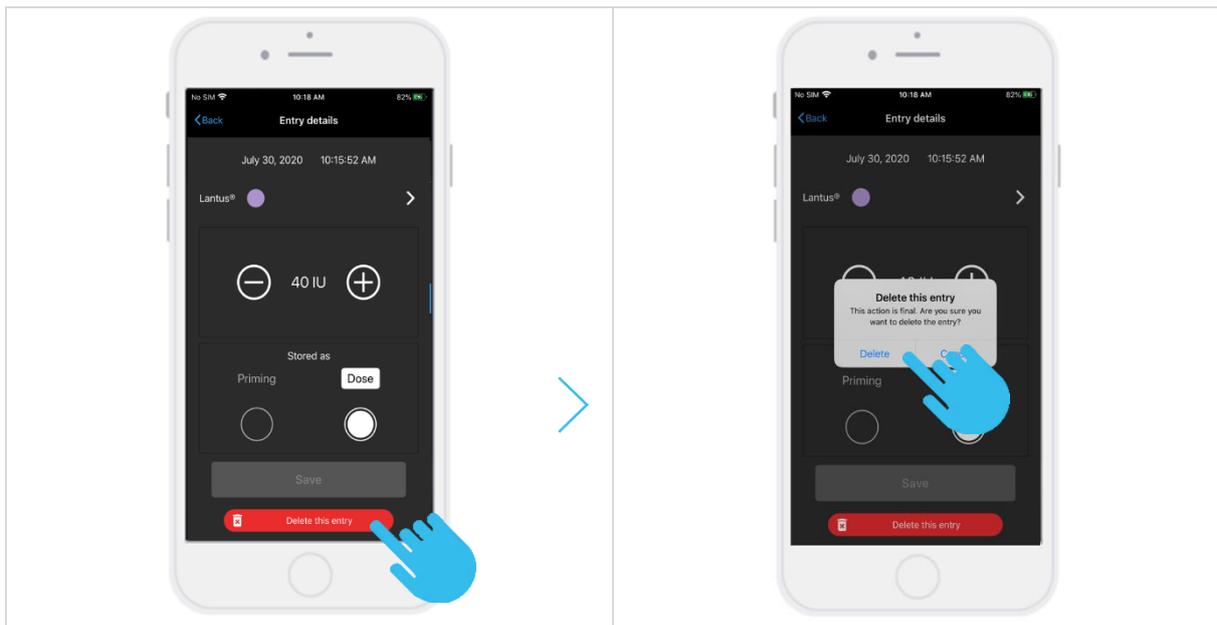

10.2.4 You can add an injection that was not recorded to the history


To add an unrecorded dose, select “Add to log”: 

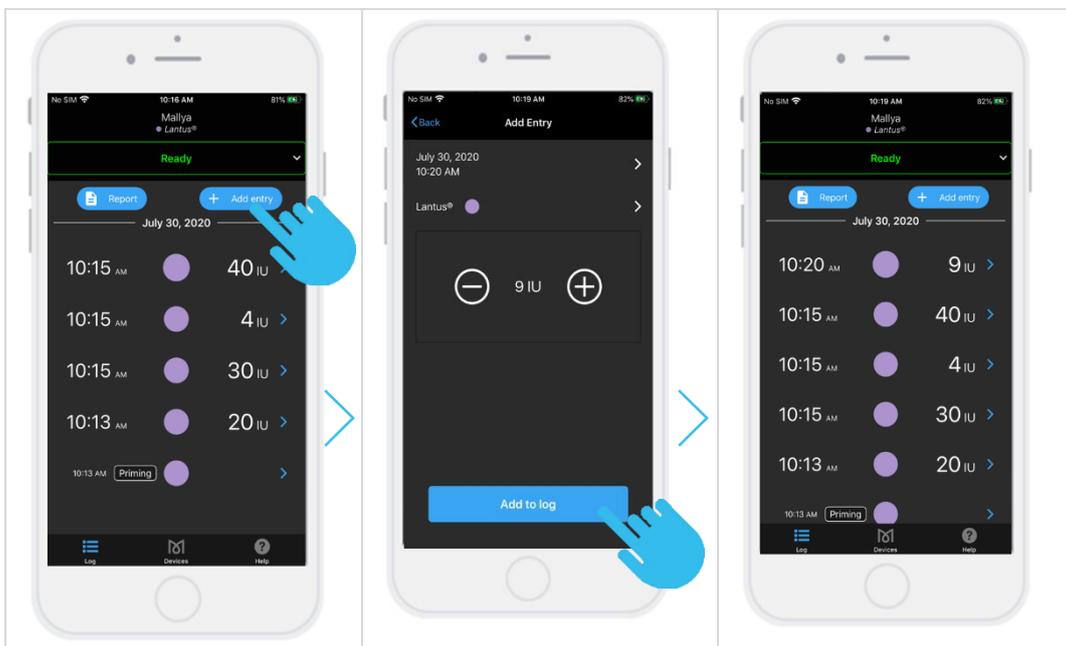

10.2.5 You can fill in a dose that was not captured by the device

If a dose was not captured, you will see the message “Dose not recorded”. You can add the dose information as follows:


Tap  to modify the information related to the injection:

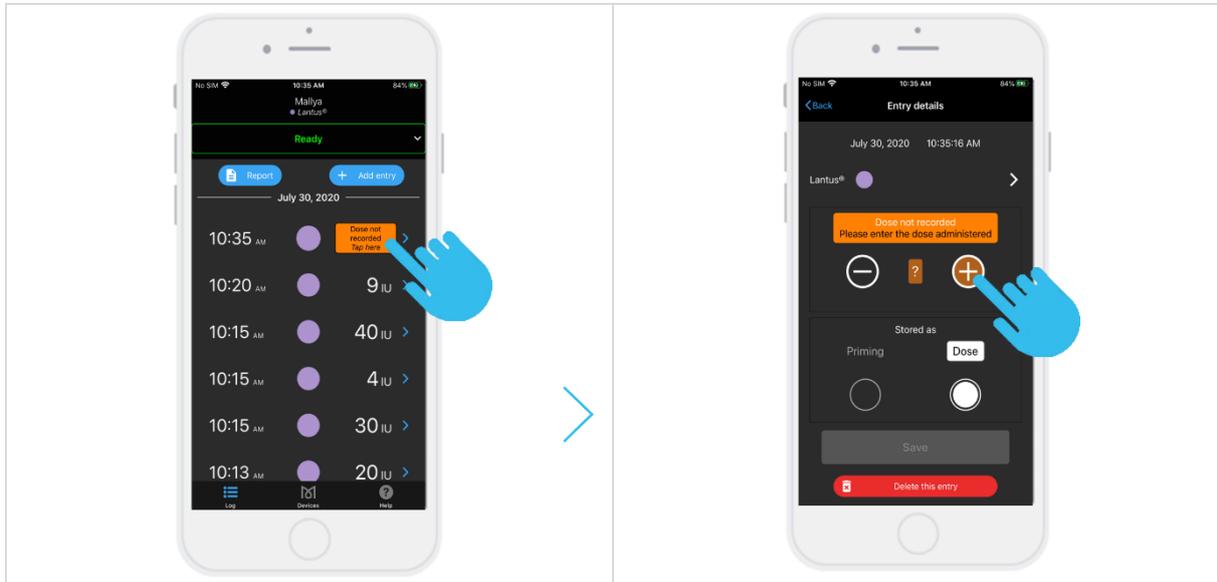


11. OTHER FEATURES OF THE MALLYA APPLICATION


11.1 GENERATE A REPORT

In the “Injections” screen, select the icon  to display the window that allows you to **generate a report**:

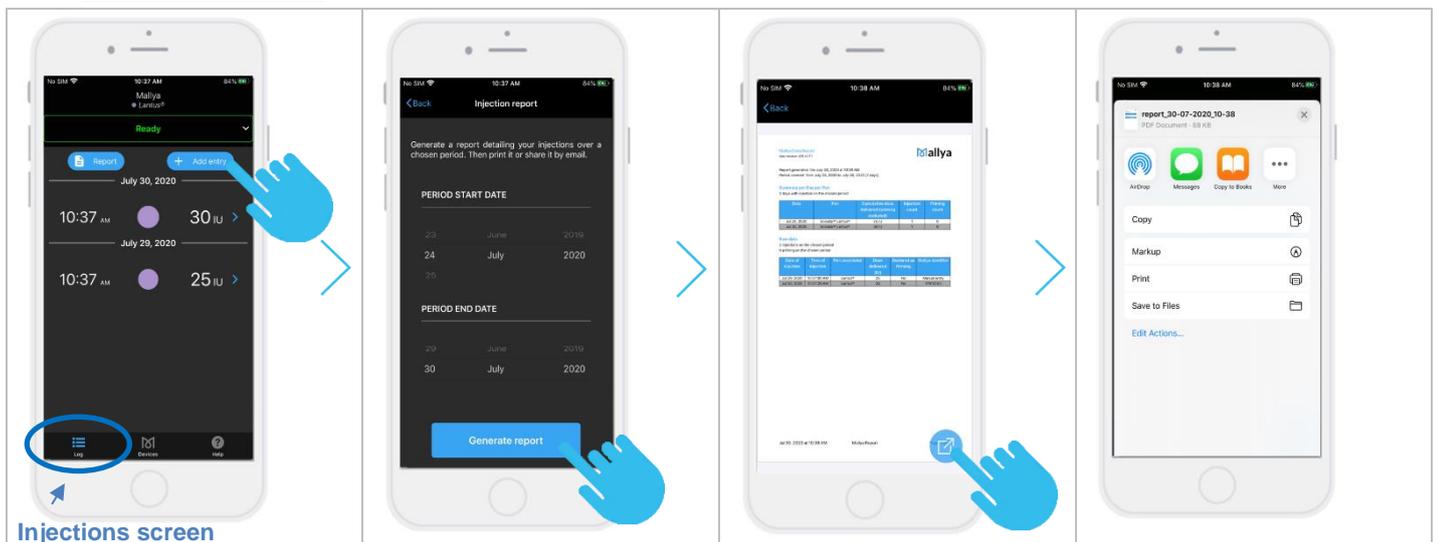

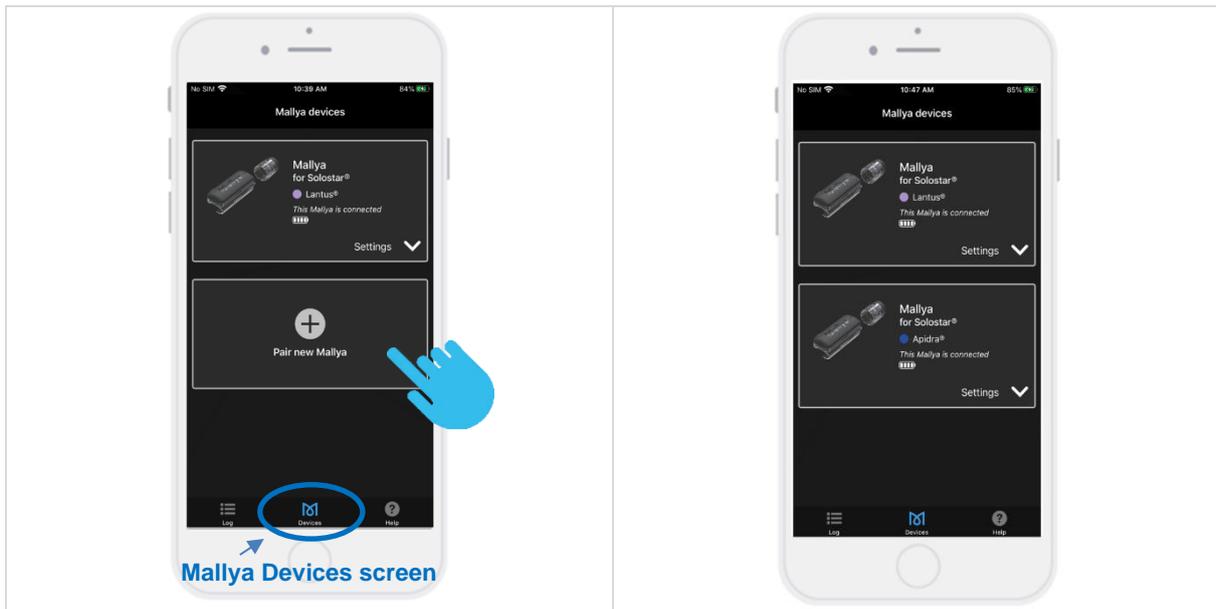11.2 PAIR TWO MALLYA TO YOUR MOBILE APP.


Both Mallya devices and associated insulin pens can be used independently. They both have the status “Ready for use”:

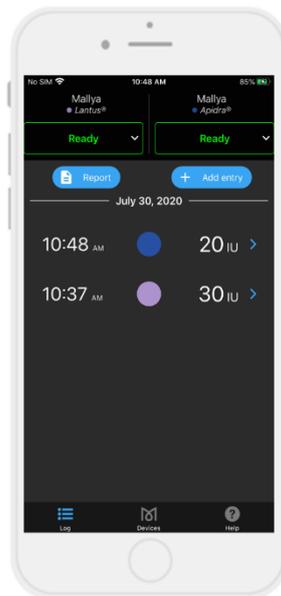**WARNING**


Make sure you do not switch the pens between the two Mallya devices. In case you have any doubt, you can use the “Find the device” functionality to identify the Mallya as explained in the step below (**step 7.3.**)

11.3 **SETTINGS**


Select , then “**Settings**” to display the screens allowing to set up the application:

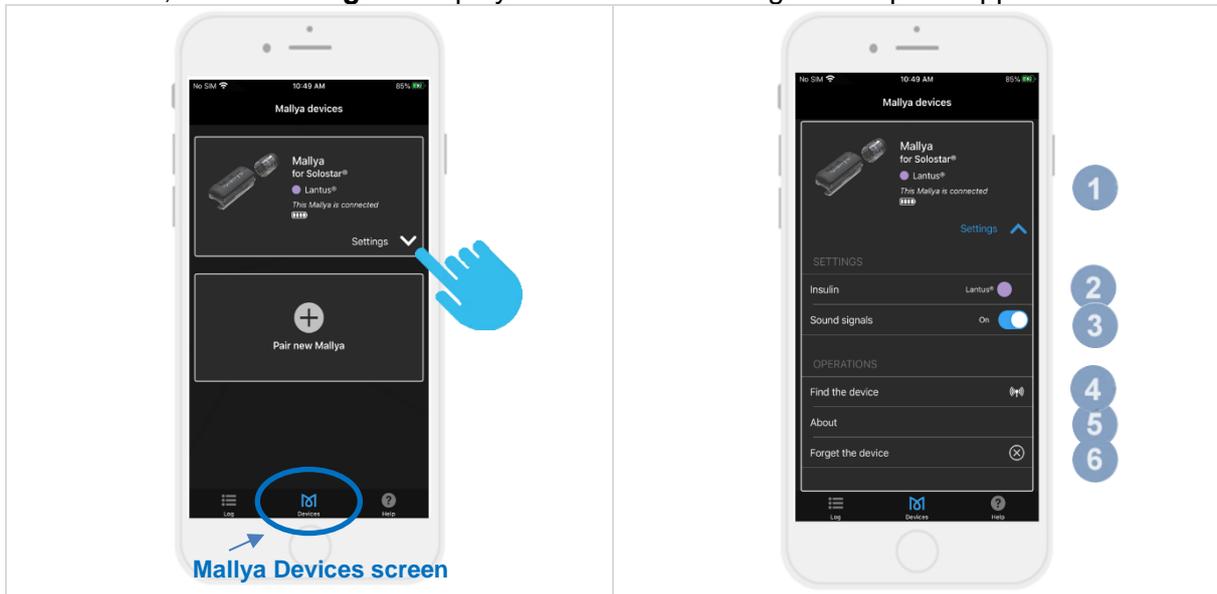

<p>1</p>	<p>Battery status : </p> <p>When Mallya is fully charged (after approximately 2 hours) : 4 bars are displayed</p> <p>Device status: <i>Mallya is connected / Disconnected since ...</i></p> <p> CAUTION A Bluetooth connection is necessary to transmit the injection data.</p>
<p>2</p>	<p>Tap “Insulin” to select the pen model onto which Mallya is currently assembled.</p> <p>Each color indicator is associated to a type of insulin pen:</p> 
<p> CAUTION</p>	<p><u>It is important to check, before each injection, that the type of insulin selected in the application corresponds to the type of insulin indicated on the label of your pen.</u></p> <p>The color indicators are also displayed in the injections screen so that you can differentiate between the types of insulin injected in the injection history.</p> <p>Example:</p>  <p style="margin-left: 150px;">Apidra</p>
<p>3</p>	<p>Sound signals: you can activate / deactivate the silent mode.</p>
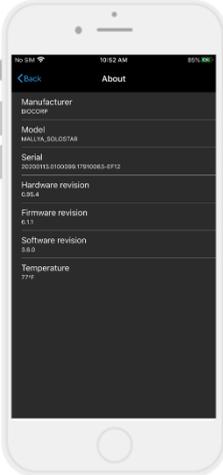
<p>4</p>	<p>Find the device: select this option to make the Mallya ring* in order to identify or locate the device. *Only if the Mallya is in STANDBY mode.</p>	
<p>5</p>	<p>“About” Information about Mallya application:</p> 	<ul style="list-style-type: none"> - Manufacturer - Model - Serial number - Hardware revision - Firmware revision - Software revision - Temperature
<p>6</p>	<p>“Forget the device” Select “Forget Mallya” to unpair Mallya.</p>	
<p> CAUTION</p> <ul style="list-style-type: none"> - If you unpair your Mallya while it is out of range, you will have to reset the system before pairing your Mallya again, in accordance with the procedures described in the step 9.3. - Reset Instructions. - Mallya has been designed to work with ONE insulin pen. If you unpair your Mallya smart base from the insulin pen, you MUST reselect your insulin pen medication. 		

11.4 “HELP” SCREEN

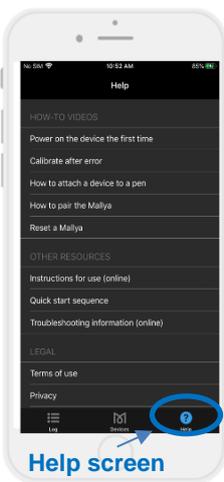
 <p>Help screen</p>	<p>Video tutorials and other resources are available from the "Help" screen:</p> <p style="text-align: center;"></p> <p>The legal notices concerning the Terms of use, the Privacy, Copyright and Version of the application (About) are also available.</p>
--	---

12. TRANSFERRING MALLYA TO A NEW PEN


12.1 REMOVE MALLYA FROM THE EMPTY PEN

Follow the procedure indicated below when your insulin pen is empty:

- 12.1.1** Remove the Mallya smart base from the insulin pen. Use your thumb to press on Mallya and to remove it from the pen.

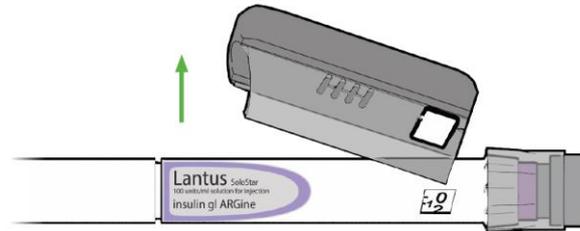

- 12.1.2** Then, remove the Mallya knob button from the pen.

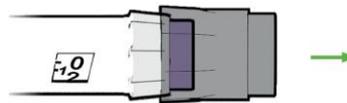

12.2 ATTACH YOUR MALLYA TO A NEW PEN

Follow **step 4.3.** on assembling your Mallya to the insulin pen.



CAUTION

- Mallya models are only compatible with the disposable insulin pens (Solostar, Kwikpen, Flexpen) they are designed for. Do not attach to, pair or use with any other insulin pen types.
- If you need to inject more than the number of units left in the pen, you may either:
 - ✓ inject the amount left in your pen and then use a new pen to deliver the rest of your dose: the Mallya application will then display 2 successive injections, or
 - ✓ get a new pen and inject the full dose: the Mallya application will display one full dose injection.







CAUTION


When Mallya is transferred onto a new pen, the application automatically detects a “**device removal**” event (eg.: your smart base and / or the knob button have / has been removed from the original pen).

In this situation, the application will require a confirmation of the pen currently assembled with Mallya.


Example:
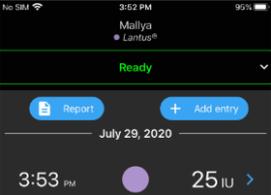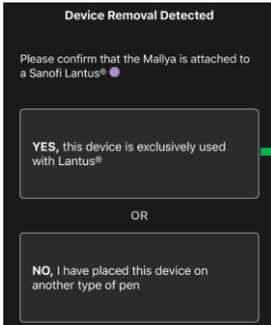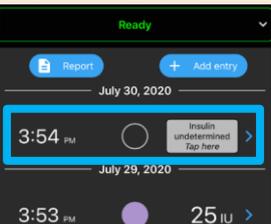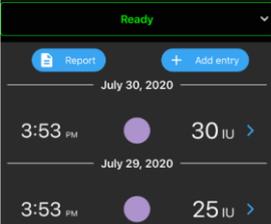
<p>12.2.1 <u>Injection screen before transferring Mallya onto a new pen:</u></p>	
<p>12.2.2 <u>Remove Mallya from the empty pen / attach Mallya onto a new pen (smart base and / or button)</u></p>	
<p>12.2.3 <u>On the next PRIMING or an INJECTION:</u></p> <p style="text-align: center;"></p> <p style="text-align: center;">A “DEVICE REMOVAL” EVENT is detected!</p>	
<p>The following screen is displayed with 2 options to choose from:</p>	
	
<p>NO, I have placed this device on another type of pen. In this case, the screens steps 4.4.3., 4.4.4., and 4.4.5. are displayed. You have to select the new pen model on which Mallya is currently assembled. Following your change of pen model, some new injections (or primings) are pending confirmation of the insulin type that has been used for the last injection. The concerned injections are shown in the list with a grey colored circled and the “insulin undetermined” text:</p> 	<p>YES, this device is exclusively used with the same type of pen (same as the original pen). In this case, the injection dose is displayed as if no “device removal” event occurred:</p> 

Select the following icon to choose the insulin type used for the injection:

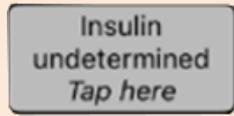

Example of the resulting screen with "Apidra" chosen:



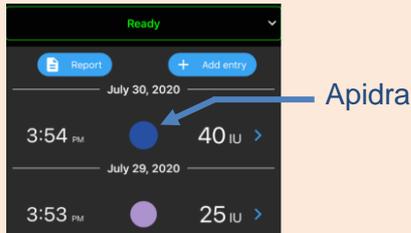


12.3 COLLECT YOUR INJECTION DATA FROM MALLYA AFTER SMARTPHONE DISCONNECTION

Your smartphone may disconnect from the Mallya system in case Bluetooth is out of range, or the battery is empty, or for other potential reasons. In this case, you can still use Mallya to record your insulin doses and, on the next connection, data will be automatically transferred from Mallya to your smartphone.

Upon re-connection, 2 options are possible:

- Your Mallya application will display the recorded injections data as usual if no “**device removal**” event occurred during the disconnection period (eg.: your smart base and / or the knob button have / has not been removed from the original pen in the meantime), or
- Your Mallya application will display the following screens in case the Mallya smart base and / or the knob button have / has been removed from the original pen during the disconnection period:

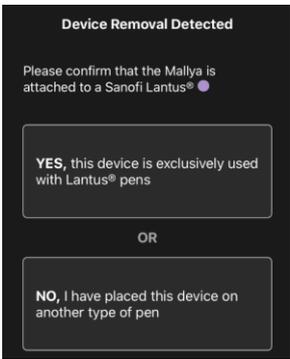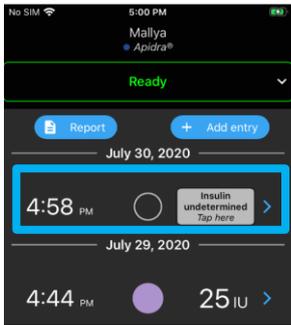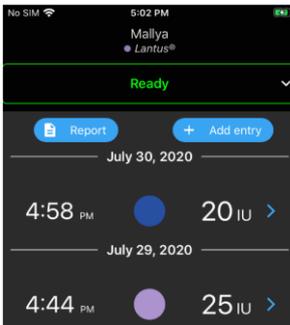“Device removal” event detected screen	Screen with undetermined injection information	Insulin undetermined dose information is completed
<p>Confirm the type of pen on which Mallya is currently attached.</p> <p>If you select “YES, this device is exclusively used with the same type of pen”: your Mallya application will display the recorded injections data as usual.</p>	<p>If you select “NO, I have placed this device on another type of pen” in the previous screen: some injections will be pending confirmation of the insulin type that was used.</p> <p>Select the “Insulin Undetermined” icon to complete the information related to the selected injection.</p>	<p>The dose quantity and insulin type color indicator are then displayed. (Apidra was chosen in the example below):</p>
		



CAUTION

Upon a first injection following a disassembly, make sure you reconnect the device with the application as soon as possible to avoid “undetermined” doses.

13. TROUBLESHOOTING



CAUTION

- Never attempt to access the internal hardware for any purpose including maintenance in all circumstances.
- When a function check is not successful and if the recommendations given in this **step 9**, do not solve the issue, you are prompted to contact a BIOCORP PRODUCTION authorized dealer.

13.1 TROUBLESHOOTING MALLYA WITH SMART BASE SIGNALS

When a visual or audible signal occurs on your Mallya, check the cause and follow the indications listed below to solve the issue:

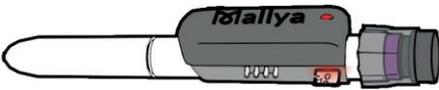
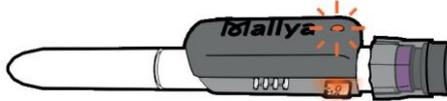
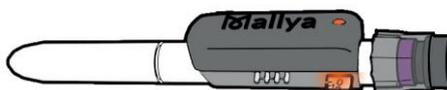
SIGNAL	CAUSE	TROUBLESHOOTING STEPS
<p>Red LED</p> 	Mallya error	<p>Remove Mallya from pen. Re-calibrate Mallya (see step 4.2.) If the issue persists: reset the system (see step 9.3.) Then, try to pair Mallya to your smartphone (see step 4.4.) If the problem is not solved, please contact a BIOCORP PRODUCTION authorized dealer.</p>
<p>Orange LED (rapid flashing)</p> 	Calibration in progress: A green light will appear for 2 seconds upon successful calibration (with a beep, then the green light will turn off).	Slowly tilt the Mallya side to side to calibrate (see step 4.2 .)
<p>Alternating orange and green flashing (rapid flashing)</p> 	Mallya device is correctly positioned onto the pen but must be paired.	Pair Mallya to your smartphone (see step 4.4 .)
<p>Orange LED (slow flashing) (every 15 seconds)</p> 	Battery is low.	Plug in and charge the Mallya smart base (see step 4.1 .)
<p>Orange LED</p> 	<ul style="list-style-type: none"> - Mallya position error on pen: the knob is not turned back to the dose "0" or - Assembling error: Mallya (smart base or button) is not correctly positioned on pen. 	<p>Make sure that:</p> <ul style="list-style-type: none"> - the knob is turned back to the dose "0" or - the Mallya window and the pen dose window are properly aligned, and - the Mallya button is correctly pushed onto the pen <p>(see step 4.3.)</p>

Table 7 - Troubleshooting with signals

INFORMATIONS REGARDING OTHER SIGNALS:

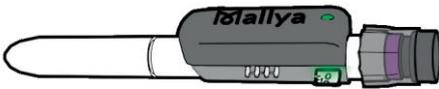
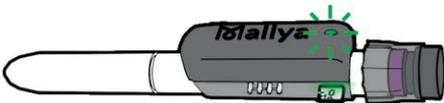
SIGNAL	CAUSE	TROUBLESHOOTING STEPS
<p>Green LED (slow flashing)</p> 	<p>Mallya is charging.</p>	<p>(See step 4.1.)</p>
<p>Green LED</p> 	<p>Mallya is properly assembled onto the pen. It is:</p> <ul style="list-style-type: none"> - either ready for use, or - the dose setting is in progress, or - the injection is in progress 	<p>N/A</p>
<p>Green LED (rapid flashing)</p> 	<p>Mallya is at the end of an injection.</p>	<p>Keep the needle in the skin:</p> <ul style="list-style-type: none"> - During the whole time Mallya's green LED is flashing, - Until Mallya emits an audible signal (BEEP) signaling the end of the injection.

Table 7 - Troubleshooting with signals (next)

13.2 TROUBLESHOOTING MALLYA WITH THE APPLICATION

This step describes the troubleshooting procedures you can perform if there is a problem with your Mallya.

Find the problem you are experiencing among those listed in the following list and apply the appropriate solution.

- If the recommended solutions do not eliminate the problem or if you encounter an unlisted problem, please contact a BIOCORP PRODUCTION authorized dealer.

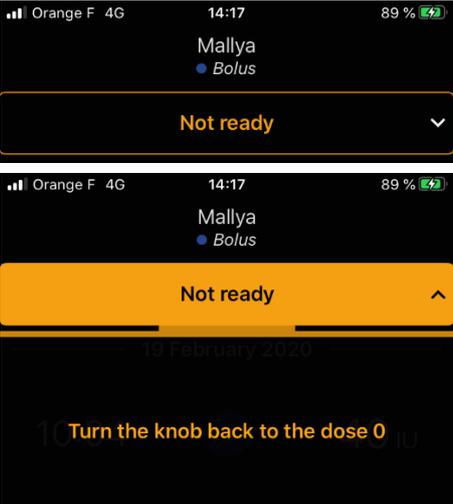
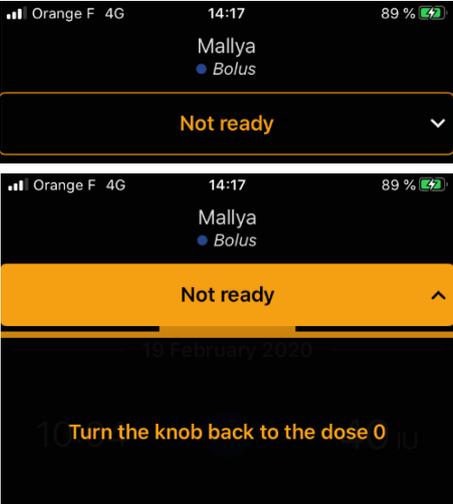
INDICATION	CAUSE	TROUBLESHOOTING STEPS
	<p>Mallya's smart base not detected</p>	<ol style="list-style-type: none"> ① Select: “Not Ready” ② Check that the Mallya device is correctly assembled on the pen (see step 4.3.)
	<p>Knob button of the pen not at ZERO</p>	<ol style="list-style-type: none"> ① Select: “Not Ready” ② Turn the knob button of the pen back to the ZERO position.

Table 8 - Troubleshooting with app.

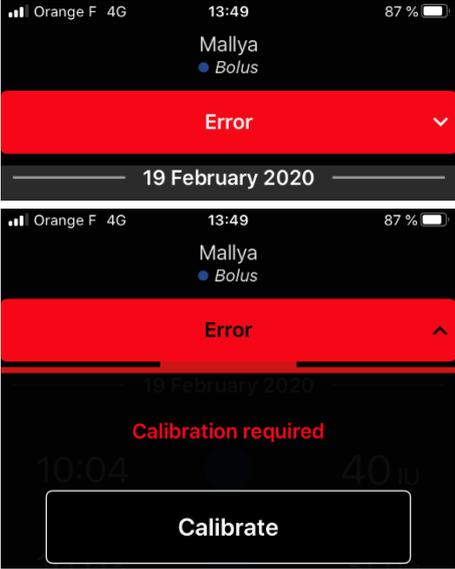
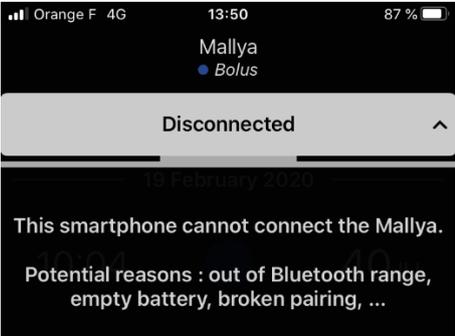
INDICATION	CAUSE	TROUBLESHOOTING STEPS
	<p>Calibration error</p>	<ol style="list-style-type: none"> ① Select “Error” ② Perform the calibration as indicated in the Mallya device Instructions for Use (see step 4.2.)
	<p>Mallya is maybe too far away from your smartphone and therefore disconnected <i>Or</i></p>	<ol style="list-style-type: none"> ① Move your Mallya device and your smartphone closer to one another.
	<p>A communication error occurred</p>	<ol style="list-style-type: none"> ① Select “Disconnected” ② Reset Mallya as explained in the Mallya device Instructions for Use (see step 9.3.)
	<p>Bluetooth issue or signal lost</p>	<ol style="list-style-type: none"> ① Check that the Mallya device is powered ON. ② Check that the Mallya device is connected to the App. ③ Check the Bluetooth connection status on your mobile phone. <p>If the problem is not solved, please contact a BIOCORP PRODUCTION authorized dealer.</p>

Table 8 - Troubleshooting with app. (next)

13.3 RESET INSTRUCTIONS

The reset procedure of Mallya may be necessary:

- If Mallya is not working properly (no response, etc...),
- If you need to pair Mallya with another mobile phone (smartphone),
- If you want to delete your personal health data from the device.

PROCEDURE:

Disassemble Mallya as indicated **step 8.1**.

Flip Mallya over so that you can see the following indications:



Take the Reset Key:



Insert the Reset Key into the hole shown below and WAIT until 3 beeps are emitted before removing the key.



3 BEEPS



Then follow **step 4**. to prepare your device for a new operation.

14. MAINTENANCE AND INSPECTION

14.1 MALLYA DEVICE LIFETIME

Mallya and accessories lifetime is 2 YEARS.

14.2 MALLYA APPLICATION UPDATES

Check each week on your smartphone (in the Apple store / Play store) that there is no Mallya application update available.

14.3 CLEANING YOUR MALLYA

Periodic maintenance of your device is not required except for cleaning.

When the surface of Mallya is dirty, follow the steps indicated below to clean it:

- ✓ Disconnect the USB cable from Mallya.
- ✓ Wipe the surface of the device with a slightly damp lint-free cloth.
- ✓ You can wipe off persistent dirt with a soft lint-free cloth slightly moistened with water containing a small amount of neutral cleaning agent.



- Disconnect the USB cable before cleaning Mallya.



WARNING

The device is intended to be non-sterile



CAUTION

- Do not wipe the surface of Mallya with chemical products or solvents (e.g. acetone) (other than those specified in this cleaning procedure) as it could lead to discoloration or deterioration.
- Please remember that the Mallya is not waterproof:
 - ✓ Do not immerse the device in water and do not clean it under running water.
 - ✓ Do not expose the device to moisture. Do not use the device in the bathroom or in areas with high temperature, humidity, dust or rain.
- Do not share Mallya with other people to prevent infection.

14.4 MAINTAIN WATER, DUST AND LIGHT RESISTANCE

As defined in the IEC standard 60529, the IP code (or Ingress Protection code) classifies and rates the degree of protection provided by mechanical casing and electrical enclosure against the intrusion of solid objects, dust, accidental contact, and water.

Mallya is rated **IP22** using this Ingress Protection rating system: your device has shown to be water and dust resistant in certain circumstances Despite this classification, your device is not at all impermeable to water damage.

NOTE:

If any liquid is found to have entered your device components (inside the sealed system): your device's warranty will be void.

Follow the next tips carefully to prevent damage to the device (operational or cosmetic issues):

- Whenever your device gets wet, dry it thoroughly with a soft, dry and clean cloth. You should dry the inside of the charging port before inserting a power connector to charge your device. If the charging port is not fully dry, your device may operate abnormally. For example, it may charge more slowly or overheat.
- Do not expose the device to direct sunlight.

- If the device is dropped or receives an impact, its water and dust resistance may be impaired.

15. TECHNICAL SPECIFICATIONS

15.1 COMPLIANCE WITH STANDARDS / REGULATION AND CLASSIFICATIONS

COMPLIANT STANDARDS	DESCRIPTION	CLASSIFICATION
IEC 60601-1	• Protection type against electric shock	Class II (double isolation)
	• Degree of protection against electric shock	Type BF
	• Mode of operation	Continuous operation
IEC 60601-1-2	• Electromagnetic compatibility	According to EN 55011 / CISPR 11 the device is classified as a group 1 device with class B emission limits.
IEC 60529	• Type of protection against the access to hazardous parts and the infiltration of water as detailed in the current edition of IEC	IP22 for the smart base

Table 9 - Compliance with standards / regulations and classifications

15.2 SMART BASE LABELLING

MALLYA SENSOR TYPE	LABELLING
Mallya smart base for insulin pen: FLEXPEN® NOVO NORDISK	
Mallya smart base for insulin pen: KWIKPEN® ELI LILLY	
Mallya smart base for insulin pen: SOLOSTAR® SANOFI	

Table 10 - Smart base labelling

15.3 SYMBOLS ON THE DEVICE AND ITS PACKAGING

SYMBOL	STANDARDS	DESCRIPTION
	IEC60417-5840	Applied part Type BF
	ISO7010-M002	Please refer to the Instructions for Use document
	ISO 15223-1	Consult the Instructions for Use documents
	ISO 15223-1	Do not use if the packaging is damaged
	ISO 15223-1	Manufacturer
	ISO 15223-1	Manufacturing date
	Symbol 5.3.4 (ISO 7000-0626) of ISO 15223-1:2012	Moisture sensitive device
	The device, packing and accessories must be disposed of separately from domestic waste in accordance with WEEE provisions for EU member countries, and with local recycling provisions and national laws.	
	ISO 15223-1	Serial Number
		Product reference
	IEC 60417-5172	Class II device (in compliance with EN 60601-1 standards)
	IEC 60417-5957	For indoor use only
IP 22	IEC 60529	Protection index: Protects persons against access to hazardous parts with fingers and against the harmful effects due to water vertically dripping against the enclosure.
	Symbol 5.3.7 (ISO 7000-0632) of ISO 15223-1:2012	Temperature limits within which the device can be safely exposed
	Symbol 5.3.8 (ISO 7000-2620) of ISO 15223-1:2012	Humidity limits within which the device can be safely exposed
	Symbol 5.3.9 (ISO 7000-2621) of ISO 15223-1:2012	Pressure limits within which the device can be safely exposed

Table 11 - Symbols on the device and its packaging

SYMBOL	STANDARDS	DESCRIPTION
	"CE marking"	Product compliant with EC regulations
	This device complies with part 15 of the FCC Rules.	Operation is subject to the following 2 conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Table 12 - Symbols on the device and its packaging (next)

15.4 ELECTRICAL RATINGS



CAUTION

Make sure you connect the Mallya USB cable to a power source compliant with the **IEC 60601-1**.

USB CHARACTERISTICS	
Input voltage	5V DC 2A
Min Voltage	500mA
Ingress Protection	IP 22
BATTERY CHARACTERISTICS	
Battery	Li-ion rechargeable : Li-ion 3,7V-130mAh
Model Number	GEB401730 / FT401235P
Operating time with fully charged battery	2 weeks
Charging time for fully charged battery	Full charge at 4,2 V for 2,5 hours

Table 13 - Electrical ratings

It is possible to connect the provided USB cable to the USB port of your PC or AC adaptor with the following characteristics:

AC ADAPTOR (NOT PROVIDED)	
Power supply	External Module with automatic adaptation of the voltage: no selection is required.
Input voltage range	90-264 Vac
Frequency range	47-63 Hz
Current input	0,5 A @ 90 VA
Category	II (double insulation)

Table 12 – Electrical ratings (next)

15.5 MATERIALS

MATERIALS	
Mallya smart base	PC/ABS (cover)
	Copolyester (body)
Mallya button	Polypropylène (body)
	POM (button)

Table 14 - Materials

15.6 ACCURACY

MEASUREMENT ACCURACY	99%
----------------------	-----

Table 15 - Accuracy

**CAUTION**

External factors may affect the dose measurement accuracy.

Factors	Affect the measurement	Does not affect the measurement
Ambient Temperature		√
Magnetic disturbance	√	
Dose Knob position	√	

Table 16 - External factors and measurement accuracy

To display the accurate doses, make sure to use the device as recommended in this Instructions for Use document:

- Follow the Warning and Caution recommendations to avoid magnetic disturbance
- Follow the information given **step 4.3.2.** to adjust the dose knob position on the pen

15.7 DISPOSAL

Dispose of Mallya according to your local guidelines.

According to Directives 2012/19/UE WEEE and 2011/65/UE RoHS II on the restriction of hazardous substances in electrical and electronic equipment on their disposal:

Public authorities adopt adequate measures to make sure that users, distributors and manufacturers contribute to the collection of electrical and electronic equipment, setting legal requirements to reuse, recover or recycle the said equipment.

**CAUTION**

You must take into account the potentially harmful effects to the environment or human health due to the improper disposal of the equipment or of parts of it.

The following graphic symbol is applied on the label of the equipment:



It reminds that all electrical and electronic equipment must be collected and disposed of separately from domestic waste at the end of life.

**CAUTION**

Before disposing of Mallya: reset the device to delete all your personal health data (see **step 9.3.**)

16. GENERAL INFORMATION



BIOCORP
 ZI LAVAU LA BECHADE
 63500 ISSOIRE
 CEDEX – FRANCE
 Tel : +33 4 73 55 70 50
www.biocorpsys.com

DISTRIBUTORS
(authorized dealers):
 See list and contact information
 at the end of leaflet

The Instructions for Use must be accessible to you at all times. You must read all instructions carefully before using the device.

If you have technical problems with our product, please contact a BIOCORP PRODUCTION authorized dealer.

The following information will be needed in order to provide you with the necessary assistance:

- **Serial Number of your Mallya unit** (this SN number can be found on the device label)

17. DISCLAIMER

The warranty is valid twelve months from the date of purchase.

The warranty covers any fault, material damage or manufacturing defect in products used in accordance with the instructions in this leaflet.

This warranty does not preclude the application of current legal guarantees under national legislation governing the sale of consumer goods.

BIOCORP PRODUCTION is not responsible for:

- Any damage resulting from disregarding the recommendations described in this Instructions for Use document.
- Any damage resulting from malfunctioning caused by a combination of connected devices.
- Any damage resulting from transport, improper use or negligence, incorrect handling, modification of the system, poor maintenance, use of incorrect voltage, lightning, infiltration of sand or water, use of parts or accessories not provided or recommended in this Instructions for Use by BIOCORP PRODUCTION.

Healthcare data protection

The terms of the regulation (EU) 2016/679 of the European Parliament and of the Council of 27 April 2016 may apply to the interactions between Biocorp Production and users of its products.

To this end, Biocorp Production agrees to comply with its obligations in accordance with the regulation applicable to Personal data protection and makes every effort to guarantee the security, confidentiality, and integrity of the Personal data that it may process.

A user has the right to access their data, rectify them, request their deletion, or exercise their right to limit the processing of such data. They may withdraw their consent to the processing of such data at any time; they may also object to the processing of their data and exercise their right to the portability of such data. To exercise these rights, please contact: gdpr@biocorp.fr

If a user considers, after having contacted Biocorp Production, that their Data Protection and Privacy rights have not been respected or that the access control provision does not comply with data protection rules, such user may file a claim online or by mail with the relevant national data protection authority.